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FAM Dispenser

Mode d'emploi

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Instructions manual

FAM Dispenser

Manual de instrucciones

Difusor FAM

DE FAM Dispenser

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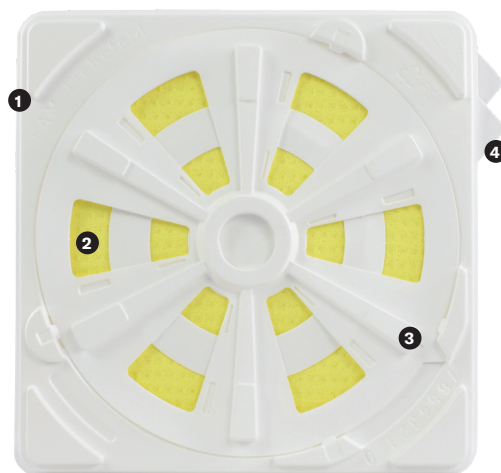
EN FAM Dispenser

For the effective treatment of varroasis of honey bees with FORMIVAR at a formic acid concentration of 70%.

Construction

- 1 Base plate with sponge cloth container.
- 2 Fungicide-free cellulose sponge cloth that optimally absorbs the acid.
- 3 Rotating disc for optimal adjustment according to the outside temperature.
- 4 Tabs for easy and safe opening of the dispenser.

The dispenser is made of polystyrene and can be used for several years. The sponge cloths can also be re-used.

How it works

The formic acid is completely absorbed by the sponge. The openings of the lid can be opened continuously by turning the disc. This allows the evaporation rate of the formic acid to be adjusted to suit the type of hive and temperature. The dispenser is placed over the brood chamber with

the openings facing downwards. The formic acid will evaporate into the colony. The formic acid vapors are heavier than the hive air and fall directly onto the Varroa. The Varroa's respiratory system is disrupted, and the mites die.

Filling and application

Separate the lid from the base of the FAM dispenser using the protruding tabs. Wash and wring out the sponge cloth. Place the base with the slightly damp sponge on a horizontal surface. For each treatment (July and September), add 130 ml of Formivar (formic acid 70%) to the sponge cloth. Then close the dispenser and latch the lid in all corners. Always keep the filled FAM dispenser with the rotating disc closed horizontally so that no acid can escape. The evaporation rate of the formic acid can be adjusted to suit the hive volume and the expected maximum

daily temperatures by adjusting the position of the rotating disc. Set the rotating disc with the pointer between 1–5 according to **the rotating disc adjustment table**. The first treatment with the FAM dispenser is scheduled for a period of 7 days immediately after the honey harvest in summer. A single feeding before the treatment is recommended. The colonies should be fully fed before the second treatment. From mid-September the FAM dispenser is placed in the colony for 14 days. Always remove the FAM dispenser from the colony after treatment to avoid contamination by wax and propolis.

Rotating disc adjustment table

	Mini-Plus Warré	CH-Kasten Zander, DNM Langstroth Segeberger National Simplex, Smith, 2× Warré	Dadant 2× Simplex AZ (Alberti Žnideršič) Commercial Deep National
1st treatment after the honey harvest for 7 days			
15–20 °C	2	3	4
20–25 °C	1	2	3
25–30 °C	first 2 days/ 2 as from day 3/	1 first 2 days/ 3 as from day 3/	2 first 2 days/ 4 as from day 3/
2nd treatment from mid-September for 14 days			
10–15 °C	4	5	5
15–20 °C	3	4	4
20–25 °C	2	3	3

Please note

- Before treatment, measure the natural Varroa mortality with the Varroa diagnostic screen for a week. Make sure the hive is well closed.
- Preferably start treatment in the morning. Always leave the hive entrance completely open.
- To reduce stress to the bees, do not start treatment immediately before thunderstorms or heavy rainfall.
- Only use after the honey harvest to avoid contamination of the honey.
- Do not treat and feed at the same time. Otherwise, there is a risk of robbery and the acid will bound to the fresh food.
- In young hives with a small hive entrance, do not close the bottom board completely. The varroa screen should be pulled back to allow better air circulation. A completely open bottom board can reduce the effectiveness by up to 20%.

Cleaning/additional information

Remove propolis with HalaApi 898. Clean wax residues from the base plate and rotating disc with APISODA and warm water. Do not put in the dishwasher, the disc will warp at temperatures above 40°C.

Formic acid treatment with the FAM dispenser has been shown to be compatible with bees in single and double story hives and in Swiss hives. Formic acid works in closed brood cells. Young brood and

bees ready to hatch are particularly sensitive to formic acid, so formic acid treatment may always result in some brood losses. These losses have no negative effect on the subsequent development of the colony.

Tip

To reduce propolisation of the dispenser, it can be placed on a 5mm wooden strip to create a head-space. **See image number 6 on the last page.**

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