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# VARROX®-Vaporizer

For efficient and well tolerated treatment against varroasis with oxalic acid dihydrate.

## Equipment and mode of action

The VARROX®-Vaporizer consists of a heater, a metallic support and 3 m of cable with 2 battery-clips. It has an output of 150 watt. The power source can be an accumulator (car battery, 12 volt, 12 ampere, capacity >40 Ah). Oxalic acid dihydrate is added to the pan, then the appliance is inserted through the entrance of the hive. As a

result of the heating action of the pan, the oxalic acid liquefies and vaporises. Oxalic acid vapour fills the hive and all the bees and surfaces are then covered with a very thin layer of oxalic acid crystals. These fine crystals are tolerated well by the bees, but have a deadly effect on varroa mites.

## Treatment

**Timing:** The use of the VARROX®-Vaporizer is particularly suitable for the follow up treatment in broodless colonies. The treatment should not be made during heavy bee flight. The temperature should be over 4 °C.

### Dosage:

One-chamber-hive (warm way, one-storey)	1 g (=1 measuring spoon*)
Two-chamber-hive (two-storey/Dadant)	2 g (=2 measuring spoons*)

\*One spoon filled loose and without pressing, corresponds to one gram oxalic acid dihydrate. Each VARROX®-Vaporizer is supplied with a measuring spoon.

**Placing:** The appropriate amount of oxalic acid dihydrate is measured into the pan. The VARROX®-Vaporizer is inserted through the entrance of the hive right below the cluster of the bees until the metallic support is fully inserted. Keep a mini-

mum distance of 2 cm to wax and bees. Remove wax from the hive floor.

**Attention:** The VARROX®-Vaporizer reaches temperatures of up to 400 °C. It can scorch the hive floor and frames if used for too long or too close. This can be avoided by using a metal sheet or wooden board below the pan. Caution with synthetic hives and plastified varroa screens. Hives or varroa screens made of plastic material must be protected with an isolation platter (minimum 4 mm thick) under the VARROX®-Vaporizer.

**Hint:** Depending on the height of the entrance and the bottom of the hive, we recommend that the VARROX®-Vaporizer is fitted in evenly by bending down both sides of the metallic support a few millimetres. The distance between the bottom of the hive and the heating element can be increased and the source of the heat is lifted from the floor.

**Sealing the hive:** Seal the whole entrance (or diagnosis drawer) around the handle of the VARROX®-Vaporizer using strips of foam or cloth.

Open floors or other openings should be shut tightly over the whole surface.

**Duration of the treatment:** After the hive is sealed up, the vaporiser should be connected to a car battery (12V) for 2½ minutes. One of the battery-clips of the cable can be constantly connected with the source of power using the contact of the second clip to switch the appliance on and off. It makes no difference which clip is connected to the plusterminal – the function of the heating is not reduced. After disconnecting the VARROX®-Vaporizer, wait for 2 minutes be-

fore the appliance is pulled out, so that the oxalic acid vapour can settle. Once removed, ensure that the entrance of the hive is shut tightly again and is left shut for another 10 minutes. After taking out the VARROX®-Vaporizer, it should be cooled in a bucket of water, so that with the next use, no vaporisation begins before the appliance is inserted into the hive.

**Swarms, artificial swarms and nucleus** can be treated at any time.

beehouse must be well ventilated. No person must remain inside the beehouse.

The VARROX®-Vaporizer is a powerful heater that generates high temperatures. The vaporiser must not be used close to flammable materials. Always put the vaporiser on a sheet of non-flammable material (i.e. metal). Never touch the metallic part of the vaporiser when in use. Caution: metallic parts may be hot for a long time after use. The vaporiser must never be switched on without supervision. After the treatment, oxalic acid crystals are found on the bottom of the hive. Caution is therefore needed if mite fall is controlled after the winter treatment. Gloves and a protective mask should be worn. The remaining oxalic acid will break down to water and carbon dioxide and will be cleaned by the bees.

## Oxalic acid-dihydrate information

Oxalic acid is a toxic and corrosive substance. When filling the VARROX®-Vaporizer and during the treatment, it is absolutely necessary to wear protective glasses, a protective mask FFP3, gloves and long-sleeved clothes.

**Occupational safety:** With treatments of aparies in the open, take care that the wind blows any escaping oxalic acid away from the user. The whole length of the cable should be used and the user should keep distance to the hive. The foam material used for sealing the hive must be handled with gloves. Avoid contact with the acid where it may have condensed on the inner side of the foam. The foam material must be washed out carefully with water at the end of the campaign. Treatments on colonies in a beehouse must only be made from outside through the entrances of the hives. During the treatment, the

## Electrical supply

The VARROX®-Vaporizer must be operated with 12 volts, 12 ampere and cannot be directly plugged into a socket of 110V or 220V. It runs on alternating and direct current (AC/DC). If the

vaporiser is connected to the car accumulator for prolonged periods the engine of the car may have to be run to avoid draining the battery.

## Warranty

Please, follow the instructions for use carefully. We do not assume any warranty for incorrect application, reasons of unsatisfactory efficacy or

possible damages that are beyond the control of Andermatt BioVet AG.

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